

Adapta[®] MT-100 ***Portable Massage Table*** ***Armrest***

SET-UP INSTRUCTIONS

TO ATTACH TO TABLE:

- **Attach the armrest hinges to the legs of the table with the four screws provided.**
- **Hinge should attach under the armrest and between the screws that connect leg panels to legs. Make sure armrest is level.**
- **Attach short strap at the center of the end frame on the inside of the table. The buckle should hang down.**
- **Thread the long strap through the slot in the armrest. The rolled, sewn part should be under the armrest.**
- **You can now buckle the straps together to adjust the height and angle of the armrest.**

TO CLOSE:

- **To close the table, unbuckle the strap and allow the armrest to drop down against the legs. Then fold your table in the normal manner.**



4717 Adams Road • P.O. Box
489
Hixson, TN 37343 U.S.A.
1-423-870-2281
1-800-592-7329 U.S.A.
1-800-361-6661 CANADA
www.chattgroup.com

